noitest

DIGITALCOL
Test

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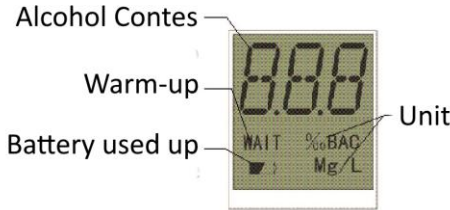
WARNINGS: The alcohol tester should be used only to give an indication of the possible presence of alcohol in the breath/blood. Everyone has a different body response to alcohol consumption so his testing results only as a reference, not a subject standard for consequent decision. You should not rely upon it as the sole basis to determine intoxication or whether it is safe to drive a motor vehicle, operate equipment, or engage in dangerous activities. **The tester must not be used as a tool for determining whether a person is able to operate a motor vehicle and machinery or device legally or safely. The intake of any alcohol will impair reflexes and judgment to operate motor vehicle.**

The test indication has no law value: the manufacturer, importer or distributor takes no responsibility whatsoever for the use of this product for any reasons.

DISPLAY

OPERATION

5. Press the power button and wait the “BIP” sound, the LCD display will undergo a countdown sequence from “10” up to “0”, meanwhile the “WAIT” signal appears.
6. When the countdown is finished the unit “BLOW”, blow one deep breath close to the intake sensor , for 5/6 seconds until another “BIP” sound; the tester should be kept to a distance of about 1 cm.
7. The numeral result will be displayed for about 8 seconds, then the unit automatically turns off.
8. Repeat always the test at least 2 times to a distance of 10 minutes to obtain a reliable result.

PRECAUTIONS

10. Insert the included batteries, respecting carefully the right polarity, otherwise the tester could be damaged.
11. To obtain accurate results, do not blow cigarette smoke or other noise gas in the intake sensor. Keep the unit away from water and other liquid fluids in a safe and dry place. Do not clean with any cleaning agent or the unit could be disturbed and damaged.
12. At the first test or after a long time storage, make 2 consecutive tests without blowing .
13. When the batteries signal is displayed on the LCD, install fresh batteries.
14. Wait at least 10 minutes before making the test after your last alcohol drinking or smoking.
15. The alcohol tester gives an indication of the blood alcohol concentration at the moment. Wait at least 1 hour after you stop drinking to obtain a reliable test; in fact blood alcohol concentration continues to rise for up 1 hour after the last alcohol drinking.
16. The alcohol tester is strictly personal, do not use as a game from more persons, the consecutive and intensive testing made by several persons avoid to obtain an accurate result and anticipate the exhaustion of the sensor.
17. In order to obtain a correct test always blow to the same intensity and distance (1 cm).

THE NOT RESPECT OF THE PRECAUTIONS BEYOND TO SUPPLY A NOT RELIABLE RESULT, IT WILL CAUSE AN ANTICIPATED LOSS OF PRECISION OF THE SENSOR.

Technical specifications	
Sensor type	Advanced NM semiconductor alcohol sensor
Measurement range	da 0,00 a 2,00 gr/l
Measurement accuracy	± 15%
Ready for operation	< 10"
Display	LCD backlight
Powersupply	2 AAA batteries (included)
Ambient Conditions	-10°C ±50°C
Dimensions	102mm x 37 mm x 20 mm
Weight	50 gr inclusive batteries

Distributed by: **NOITEST srl**
 Via Flaminia 216/B - 61043 - Pesaro - Italia
 www.noitest.it info@noitest.it
 made in PRC